



FS180-HC Face Shield / *Personal Protective Equipment (PPE)*

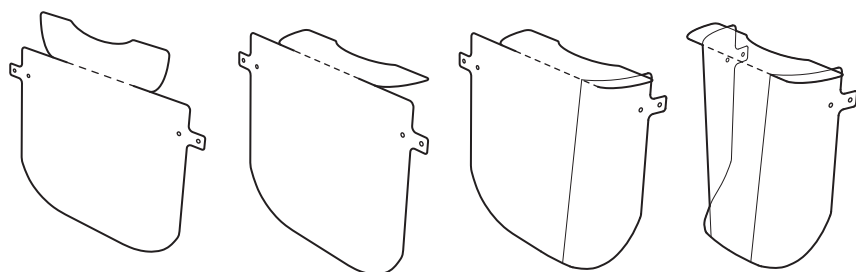

ACRO Manufacturing Industries Ltd. holds an active Health Canada Class I Medical Device Establishment Licence for manufacture (License #12370) and is ISO 9001:2015 QMS Certified

Copyright 2020 Jonathon Anderson. All rights reserved. Patent pending.

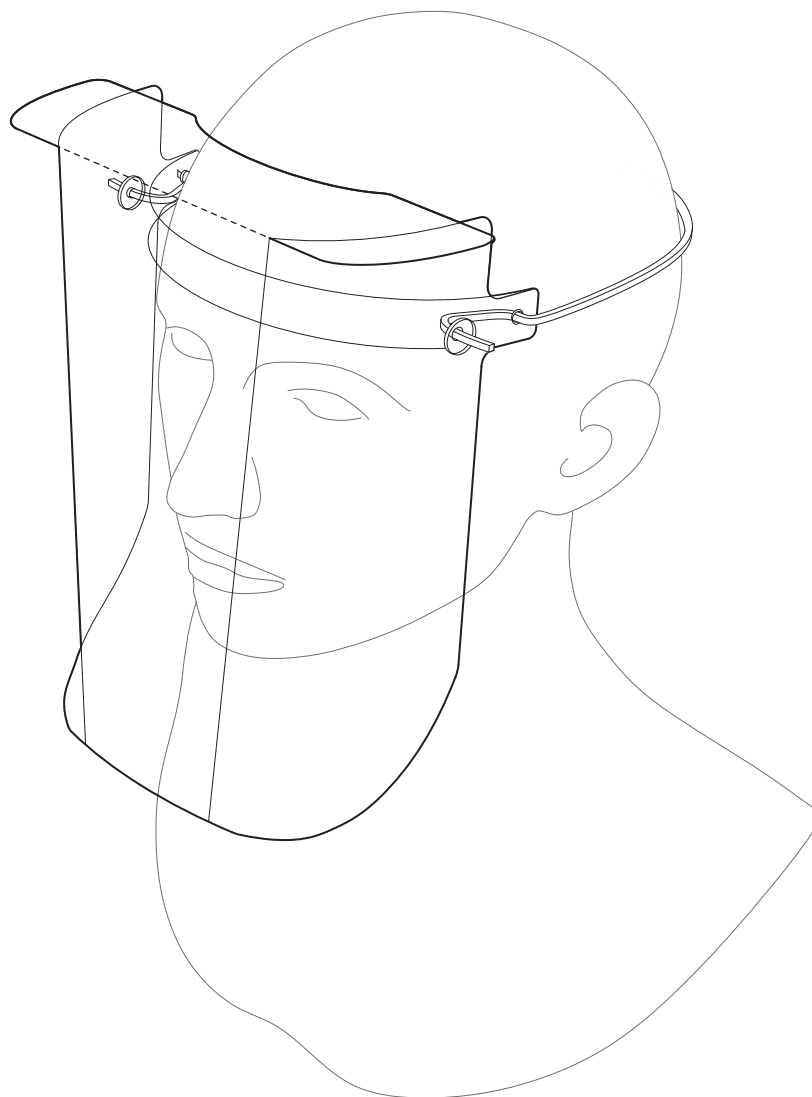




FS180-HC Face Shield / *Personal Protective Equipment (PPE)*

The FS180-HC Face Shield is a medical-grade device used to protect an individuals' eyes, nose, and face from transmission of COVID-19 via droplet exposure. Each shield is comprised of three parts—face shield, forehead band, and elastic strap—and is lightweight, designed for multi-directional droplet abatement, fits a variety of head sizes, and may be sanitized for reuse.

- + No Fog Feature
- + Tested by Infection Prevention and Control (IPAC) at Unity Health Toronto
- + May be sanitized and reused for extended lifecycle
- + Clearance for simultaneous use of N95 mask and goggles
- + Adjustable to fit various head sizes
- + Extended length and sides for droplet protection
- + Lightweight, extreme comfort for extended wear
- + Ships flat; Assembled by end user




Assembly Instructions

- 1 Lay all components on a clean surface REMOVE PROTECTIVE FILM
- 2 Align the face shield with one side of the forehead band

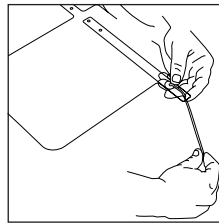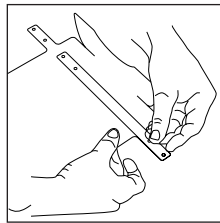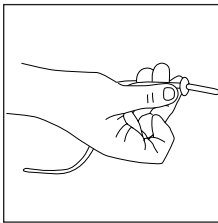

3 Assemble the first side of the face shield

- a Thread a washer on one side of the head strap
- b From outside of the shield, thread the washer and head strap through the inner hole
- c Return the strap through the outer hole to create a secure loop and pull taut

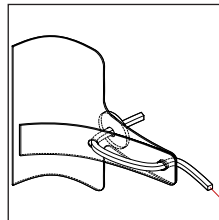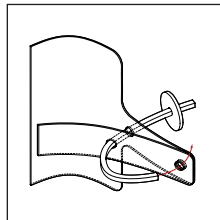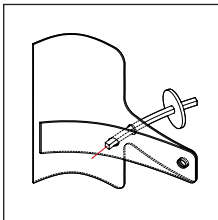


4 Assemble the second side of the face shield

- a From outside of the shield, thread the washer and head strap through the outer hole
- b Return the strap through the inner hole to create a secure loop and pull taut
- c Thread a washer on the side of the head strap


5 To improve fit and comfort, hold the washer and pull the head strap to the side

ACRO Manufacturing Industries Ltd. holds an active Health Canada Class I Medical Device Establishment Licence for manufacture (License #12370) and is ISO 9001:2015 QMS Certified

Copyright 2020 Jonathon Anderson. All rights reserved. Patent pending.

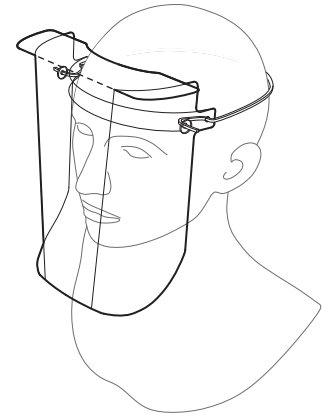

Components

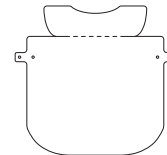

Shield
0.020" PETG



Forehead Band
0.020" PETG



Head Strap
1/8 " x 1/8" Neoprene



Washer x 2
5/8" (outer diameter) Silicone

